



5 SIMPLE STEPS TO EFFICIENT USE

Dual chambers designed to make pouring, counting, and dispensing every pill seamless.

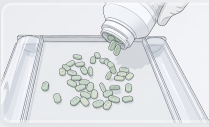


1 Pour pills onto the Dumping Tray

Pour your target pills onto the center of the Dumping Tray area for counting.

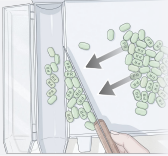


2 Move excess pills to Left Chamber

Use the spatula to carefully slide any extra pills into the Left Chamber. Move them until the display confirms your target count.

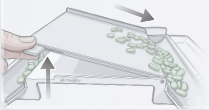

3 Slide counted pills to Right Chamber

Lift the Dumping Tray holder and slide all pills into the Right Chamber.

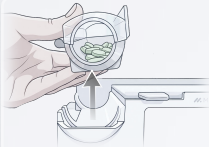

4 Return extras from Left Chamber

Detach the Left Chamber and pour remaining pills back into the original container.



5 Dispense from Right Chamber

Detach the Right Chamber and pour counted pills into a new vial.

Done!



Alcohol-resistant material for easy cleaning

Simply wipe with an alcohol swab or a soft cloth with disinfectant. Always ready for hygienic use between patients.





Need Support? We're here for you

Having any issues with your shipment or questions about usage?


Scan the QR code to contact us.



PILLEYE DOUBLE-SIDED TRAY

Best Practices for Accuracy & Usability

Load the Dumping Tray Within Capacity

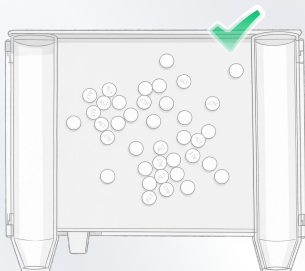


Do not exceed 3/4 of the tray. For larger quantities, split into multiple counts.

Spread Pills Evenly Before Counting




Detach the tray from your PillEye Station or Prism Kit, then gently shake it in all directions to spread pills into an even, flat layer. Reattach before counting.

Use the Spatula for Precise Counting

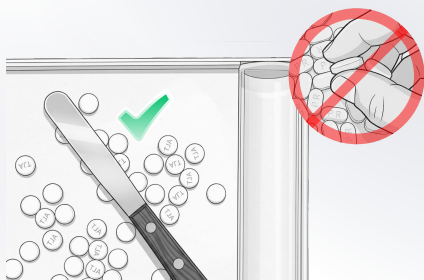

Use the included spatula to separate pills and reach your exact target count — hygienically, without touching pills directly.

Be careful with the Dumping Tray Angle

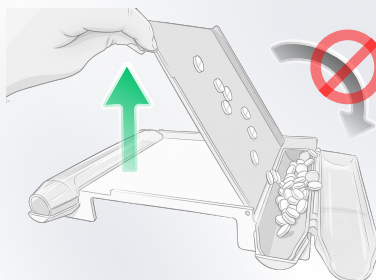

The dumping tray has a maximum tilt angle. Do not force it beyond this point — it may cause spills or damage.