

Getting Started with PillEye Prism

We're excited to get you started 😊

Start experiencing the power of your PillEye Prism today

1. Set up your iPad
2. Launch your PillEye app
3. Assemble your PillEye Prism.



Package Contents - Please check all three separate boxes are included as follows



PillEye Prism



PillEye Double-sided Tray




iPad

How to Set Up Your PillEye Prism



Installation Video

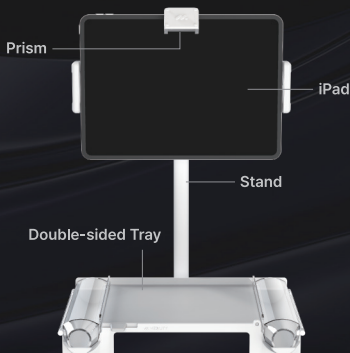

Pro Tips

Start experiencing fast and accurate pill counting with PillEye Prism

- For the most accurate results, spread the pills evenly across the tray with larger quantities a gentle shake can help you quickly achieve an even spread
- You can adjust the height of the stand to increase the camera's field of view. This allows you to count a larger volume of pills on the tray more efficiently

1

Quick Start Guide: Your New iPad

Your iPad comes pre-configured and ready to use

- Unbox your new iPad and power it on
- Unlock it with the iPad passcode we provided with your order.
- On the Home screen, select the PillEye App.




PillEye App

2

Stand Assembly Guide


- Remove the Prism and Stand from the package
- Find the **Stand base, Stand head, M5 bolt, and Allen key**
- Insert the Stand head into the base and **use the Allen key to fasten the bolt from the bottom**, securing the head to the base

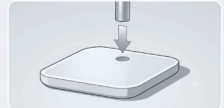

3

Mount Prism

- **Align the iPad's front camera with the center of the Prism**, then carefully place the Prism onto the iPad
- **Tighten the knob on the back of the Prism** until it is securely locked
- Check the PillEye app camera view and adjust the knob as needed

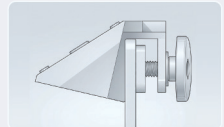

4

Place Double-sided Tray & Spatula

- Remove the Double-sided Tray and Spatula from the package
- Securely place the Double-sided Tray onto the Stand base
- Check the PillEye app camera view to confirm the Double-sided Tray is clearly visible

Contact Us

If you experience any of the following

- Difficulties with setup or app usage
- Missing iPad passcode information
- Issue with your shipment

Please reach out to us at our Customer Support

