

PillEye Station

Quick Start Guide

Follow these steps for a quick setup.

PillEye Station arrives pre-configured and ready.

Start counting with precision in just minutes.



OVERVIEW



1 iPad Dock

Securely holds and positions the iPad for intuitive operation

2 Imaging Module

High-resolution camera for real-time pill detection via Vision AI

3 Tray Dock

Magnetic docking for stable Double-sided Tray placement

4 Power & Connectivity Panel

USB-C, USB-A, Ethernet for flexible connectivity

PACKAGE CONTENTS



PillEye Station



iPad



Double-sided Tray



Spatula



Accessory Box Power Adapter & USB-C Cable


QUICK SETUP IN 5 STEPS

No assembly required. iPad and PillEye app are pre-configured and ready to go!

1 UNBOX & PREPARE

Open the box, take out everything in the box


Take out the PillEye Station, Accessory Box, and Double-sided Tray box.

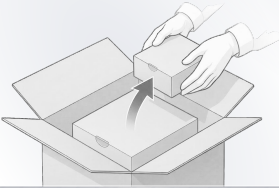

2 CONNECT POWER

Connect Power Adapter and USB-C Cable


Take the Power Adapter and Cable from the Accessory Box. Connect them to the Power & Connectivity Panel and plug the adapter into an outlet.

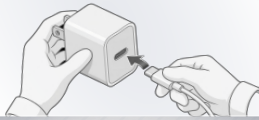

3 PLACE DOUBLE-SIDED TRAY

Double-sided Tray place it onto the Tray Dock

Unbox the Double-sided Tray and place it onto the Tray Dock, aligning it securely in position.

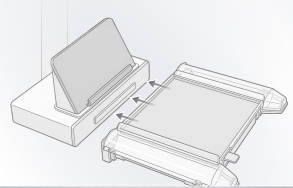

4 MOUNT IPAD & CONNECT

Place iPad on iPad Dock, and connect PillEye Station

Place the iPad on the iPad dock and connect the USB-C cable from the dock to the iPad's USB-C port.




Your iPad is all set! just power on

Pre-configured with PillEye App
— just power on and start counting

5 POWER ON & VERIFY

Turn On iPad & Open PillEye

Turn on the iPad and open the PillEye B2B app. Verify you see the counting screen as shown below.

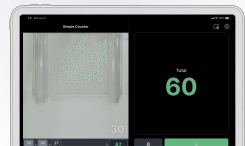

Need Support? We're here for you

Having any issues with the shipment or setup? Scan the QR code to contact us.



Explore PillEye Station Guides & Tutorials

Scan to watch PillEye Station walkthroughs and tutorials.

